


Self-injecting with Gonal-f[®] RFF Pen—4 easy steps

1 Attach needle



Remove pen cap and wipe threaded tip with alcohol swab. Remove peel tab from outer needle cap. Press threaded tip of pen into open end of outer needle cap. Twist pen clockwise until needle is securely attached. Remove outer needle cap.

2 Set the dose

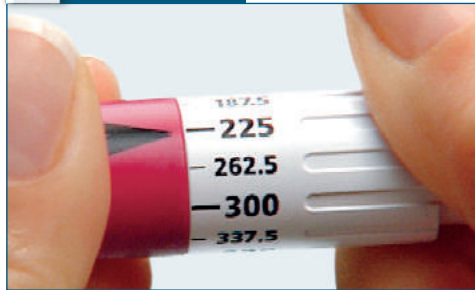


Turn dosage dial so your prescribed dose is lined up with the black dose arrow.

● **Recheck that correct dose on the dosage dial is lined up with black dose arrow before proceeding.**

If the dialed dose is too high or too low, reset before pulling the injection button.

3 Load and confirm

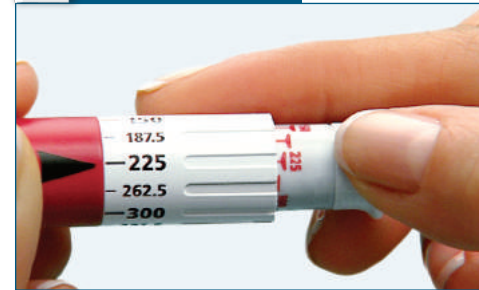


Load the dose by pulling out injection button as far as it will go. Find the last fully visible flat red arrow on the injection button. It confirms the dose that is loaded and ready to be injected. (If the loaded dose is lower than the prescribed dose, there is not enough drug to complete the dose. Use a new Gonal-f[®] RFF Pen to complete dose. See back of this page for more detailed instructions.)

4 Inject



Note: Read the instructions prior to injection. Clean recommended injection site with alcohol swab and let air-dry. Remove inner needle cap. **Do not touch needle or allow it to touch any surface.** Inject needle into skin at a 90° angle. Push the injection button until it stops clicking. After the last click, keep the needle in the skin for at least 5 seconds. Remove the needle and apply pressure as needed with a gauze pad.

Priming the pen: First use only

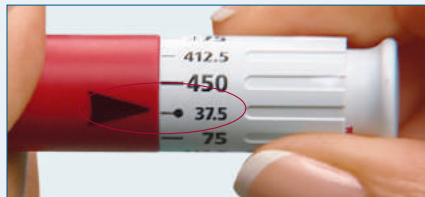


Follow instructions for Step 1: Attach needle. Set dose arrow at 37.5. Pull out injection button as far as it will go. Remove inner needle cap and hold pen so needle points upward. Tap prefilled reservoir to let any air bubbles rise to the top. (It's normal if a few small air bubbles remain.)



Keep needle pointing upward and push in injection button completely. Hold until you hear a click. A small amount of liquid should appear from needle tip. (If no liquid comes out, reprime the pen until it does.) Replace inner needle cap.


Disposal

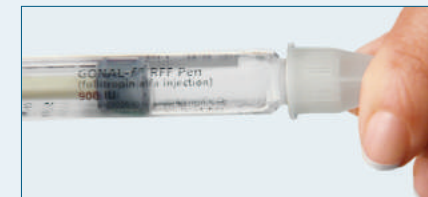


Hold pen firmly by drug reservoir and replace outer needle cap. Grip outer needle cap firmly, and unscrew pen from needle by turning pen counterclockwise. Dispose of needle in safety container.


Storage

After use, replace pen cap and store pen at room temperature or in refrigerator for up to 28 days.

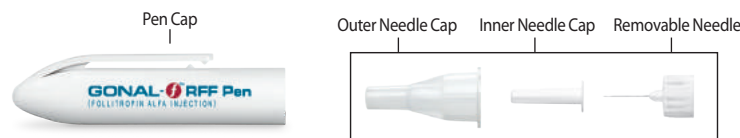

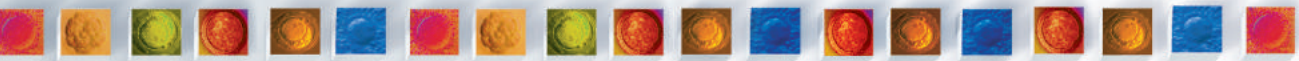

When you need to complete a dose with a new pen...

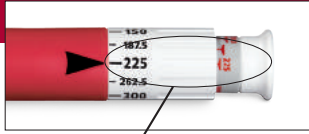

When you pull out the injection button, if the last visible flat red arrow **shows a number less than the prescribed dose**, you do not have enough medication to complete your prescribed dose. In this case, follow the directions to the right.

1 In the red-shaded area of the chart, find the dose remaining in your current pen.

Dose Left in Current Pen

37.5	75	112.5	150	187.5	225	262.5	300	337.5	375	412.5

2 Find your prescribed dose in the column below.

Prescribed Dose	450	412.5	375	337.5	300	262.5	225	187.5	150	112.5	75	37.5
	412.5	375	337.5	300	262.5	225	187.5	150	112.5	75	37.5	
	375	337.5	300	262.5	225	187.5	150	112.5	75	37.5		
	337.5	300	262.5	225	187.5	150	112.5	75	37.5			
	300	262.5	225	187.5	150	112.5	75	37.5				
	262.5	225	187.5	150	112.5	75	37.5					
	225	187.5	150	112.5	75	37.5						
	187.5	150	112.5	75	37.5							
	150	112.5	75	37.5								
	112.5	75	37.5									
	75	37.5										

3 Find the intersection of "Dose Left in Current Pen" and "Prescribed Dose." The number at this intersection indicates the dose you need to dial on a new pen to complete your prescribed dose.



24/7 compassionate listening, educational information, and medication support.

Toll Free 1-866-LETS TRY (1-866-538 7879)
www.fertilitylifelines.com

EMD Serono Fertility Medication Instructional Videos available at
www.fertilitylifelines.com